



GL5 | GLS5

Floor Lifters

Guldmann™

GL5 mobile lifter



Making transfers easier, reducing the risk of back injuries to care providers, and improving the comfort and safety for both the carer and user

The GL5 mobile lifters are the floor-based alternatives to the ceiling mounted lifting solutions offered in the Guldmann range of products.

We have been listening to our users' advice and feedback for more than a quarter of a century, creating the next generation of mobile lifters based on what you, the carer, and the user want and need. And we are still doing it.

We believe in quality, simplicity and attention to detail to provide products that create more "Time to Care".

What makes the GL5 mobile floor lifters unique is the attention to detail that makes the lifter easy to use and provides comfort and a safer lifting process.



GL5 – strength comes from the inside

The characteristics of GL5

Low reach – ability to lift directly from the floor

High reach – ability to lift tall users in large slings, without difficulty

Flexible – with the appropriate type of sling / accessories the GL5 can deal with all moving and handling requirements

User friendly – easy for both tall and short people to use. Can be operated either by the hand control or by the operating panel

Reliable – the GL5 is strong, stable, reliable and safe – designed for intensive use in professional working environments

Intelligent – latest technology that provides access to statistics via SIC (service information console) showing

- Number of lifts
- Number of lifts since last service
- Life span of components (actuator, battery)
- Reports of errors

The GL5 also features a visual service indicator and is prepared for future software updates

Modern design – offering on or off board charging, to suit to environment in which the GL5 is used. The GL5 has been matched to the “Red Dot” award winning design of the GH3 ceiling hoist / system


All round capabilities – the GL5 makes it easy to deal with a wide range of moving and handling situations while being ergonomically sound and safe for both the care provider and user.

Both models feature electric lifting and width adjustment to allow easy access around chairs, baths and the ability to lift from the floor.

There are many different functions that enable GL5 mobile lifters to meet the needs and requirements encountered in the professional care environments, where lifting equipment is required. The carefully configured design ensures there is always an abundance of working space for the care provider to perform a safe and comfortable lift for the user.

The configuration means lifting capacity is more than adequate, and GL5 lifters are very stable. These lifters have the ability to be charged on/off board to suit various working environments. The low friction castors, ergonomically designed push handle and the lifting design make the lifters easier to manoeuvre. User friendly features include easy to use buttons and indicators and safety features with automatic on/off and easy to reach emergency stop / lowering functions.

The GL5 lifter is available as two different models, with the lifting capacity of 304 lbs (155 kg) or 450 lbs (205 kg), respectively.

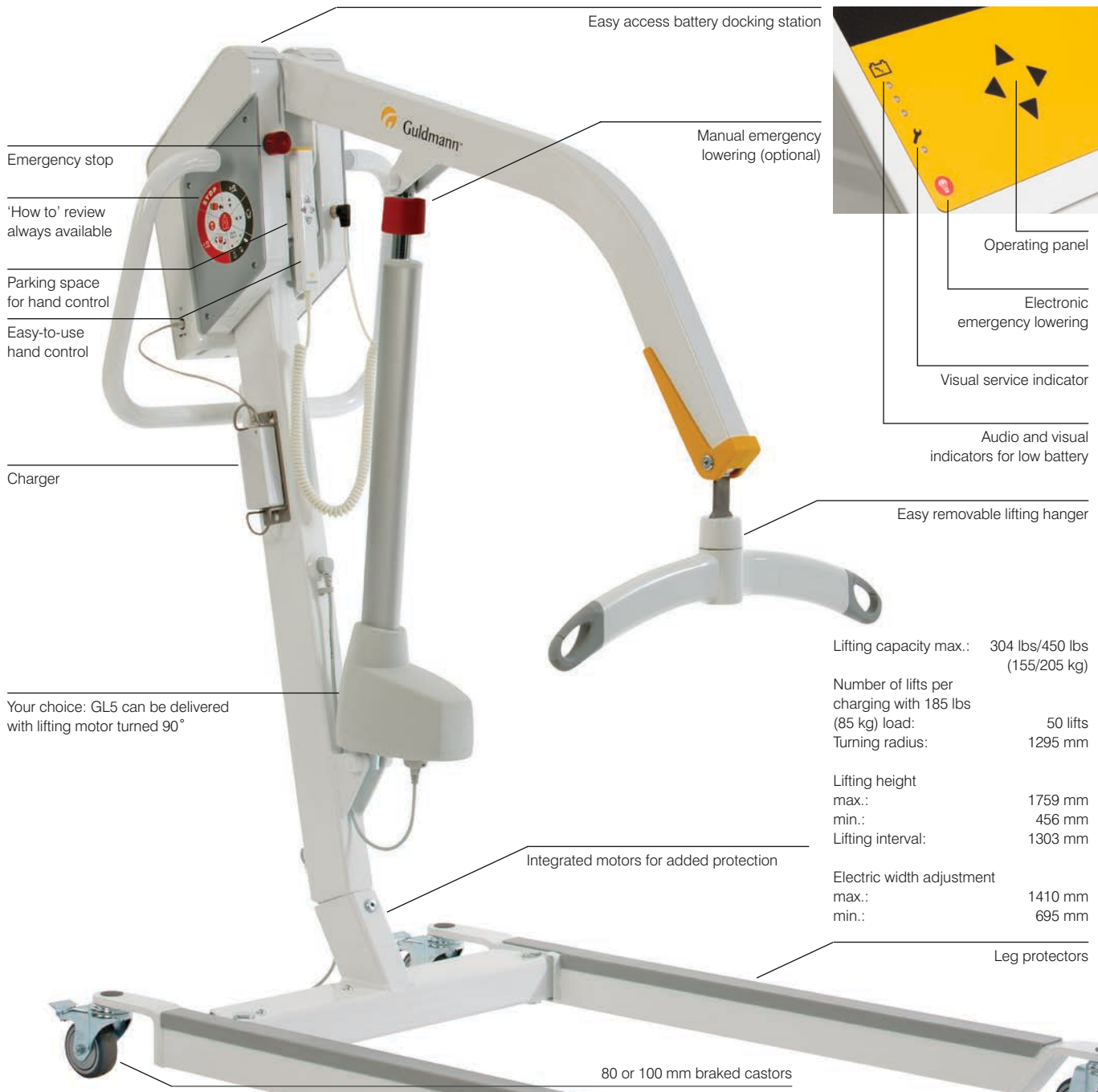

Easy access battery docking station

Emergency stop

'How to' review always available

Parking space for hand control

Easy-to-use hand control

Charger

Your choice: GL5 can be delivered with lifting motor turned 90°

Integrated motors for added protection

Manual emergency lowering (optional)

Operating panel

Electronic emergency lowering

Visual service indicator

Audio and visual indicators for low battery

Easy removable lifting hanger

Lifting capacity max.: 304 lbs/450 lbs (155/205 kg)

Number of lifts per charging with 185 lbs (85 kg) load: 50 lifts

Turning radius: 1295 mm

Lifting height max.: 1759 mm

min.: 456 mm

Lifting interval: 1303 mm

Electric width adjustment max.: 1410 mm

min.: 695 mm

Leg protectors

80 or 100 mm braked castors


GL5 handles all the lifting tasks



Lifting to and from the floor

GL5 can lift directly from the floor to a chair or bed. This makes a GL5 the ideal solution if an unforeseen lifting need occurs where there is no ceiling mounted hoist available. The lower section of the lifter is designed to give the user ample room and the considerable reach of the lifting arm means the user doesn't feel boxed in.


GL5 is ideal for use in physiotherapy, where users have to be lifted from, for example, a chair down onto an exercise mattress. The lower section can be placed in the wide position to make sure there is enough room to make the lift directly above the mattress.

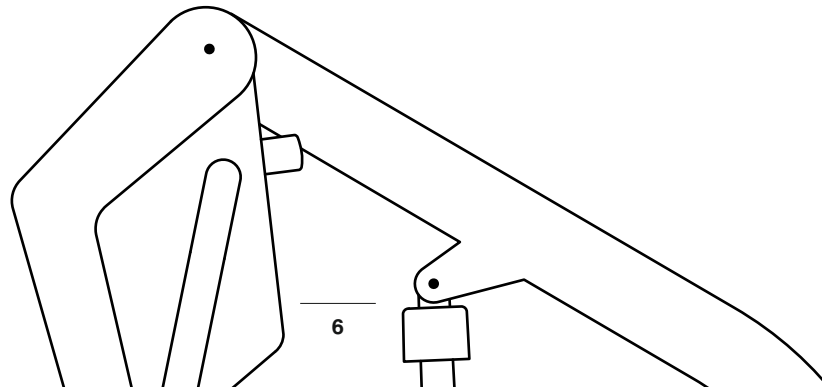


“Mobile lifters are particularly suitable in places and situations where there is no ceiling-mounted hoist system within reach. Maybe there is no room for a stationary installation. Maybe the place does not normally involve lifting. Whatever the reason, a mobile lifter is a practical and cost-effective lifting aid and a great means of extra safety”

Safe patient handling carer



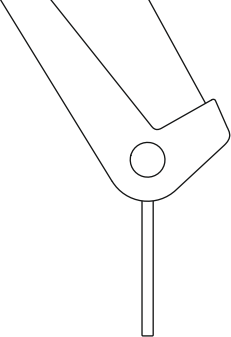

From day to day routines...

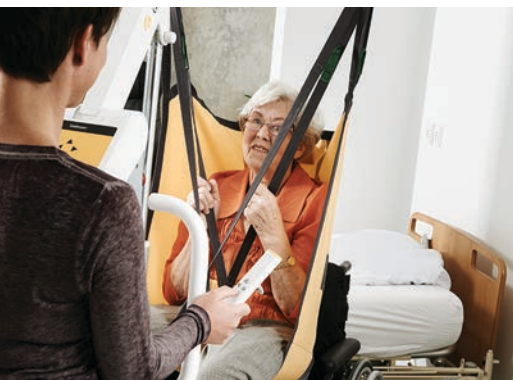


Lifting to and from the bed

GL5 ensures effective lifting to and from the bed. The ability to lift high and the long reach result in the user being placed correctly into the middle of the bed, or be positioned correctly into a wheelchair.

“This mobile lifter makes it easier for both users and carers to deal with daily situations involving lifting and moving. It may well help prevent work strains and back injuries for our care staff”

Manager, Long Term Care Facility

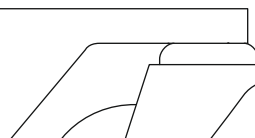



...to special demands




Heavy lifting

With the capacity to lift 450 lbs (205 kg) the GL5 is ideal for lifting bariatric patients. It has ample room for the user, a high lifting capacity and exceptional stability that leaves the user without any discomfort or anxiety.

Horizontal lifting

With the advanced design of the lifting height and reach the GL5 is well suited for horizontal lifting with the Guldmann stretcher style accessories.

The GL5 mobile lifter is an all-round lifter that meets the majority of moving and handling requirements when used with the Guldmann range of slings and accessories

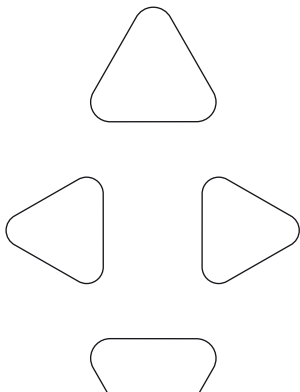



Guldmann



GLS5 Active lifter



One lift for many functions

- Sit to stand
- Standing rehabilitation (with foot tray removed)
- Transfers from chair to chair
- Transfers from bed to chair
- Transfers from bed/chair to toilet

Like the GL5, the GLS5 Active lifter is a mobile lifting device that is designed to ease everyday routines for both user and carers. The GLS5 raises the user from sitting to standing position and is also useful for the frequent daily transfers between bed, chair and bathroom.

Flexible to assist a wider range of users

To be able to use the GLS5, the user must have at least partial control of his or her upper body and have the ability to co-operate. When standing in the GLS5 the users' feet are placed on the foot tray while the legs are supported by a padded and adjustable knee support. For patients that require additional support the calf support strap can be used. The GLS5 is fitted with multiple sling attachment points that allow the active sling range to be used to create optional lifting positions for a wide range of patients. To facilitate walking, the foot tray can be removed if the user has good standing balance.

An easy, comfortable and safe way to raise and support users with partial or complete control of upper body



The characteristics of GLS5



Easy change of battery

Easy to maintain


- Easy changeable battery for possible external charging
- Removable foot tray, easy to clean
- Visual service indicator
- Prepared for future software upgrades
- Access to statistics via SIC (Service Information Console):
 - Number of lifts, lifespan
 - Number of lifts since last inspection
 - Number of battery warnings
 - Number of errors


Easy to operate, safe to use

For the carer the GLS5 is simple and easy to operate. Lifting and lowering are managed electrically, as is the adjustment of the leg width. The push handles are ergonomically profiled and designed to fit most heights and sizes. To help with manoeuvrability the lifter is fitted with a rear foot push bar. The GLS5 can be operated by the hand control or by the operating panel.

Safety has been a primary concern in Guldmanns' development of the GLS5. It is fitted with an emergency stop button which is easily accessible and can be released manually. Electric or manual (optional) emergency lowering, battery warning – visual and audio, actuator replacement indicator, ability to obtain the usage information from the lift using our SIC interface.

Finally, the user guide with a “how to” review is located on the side of the control box.

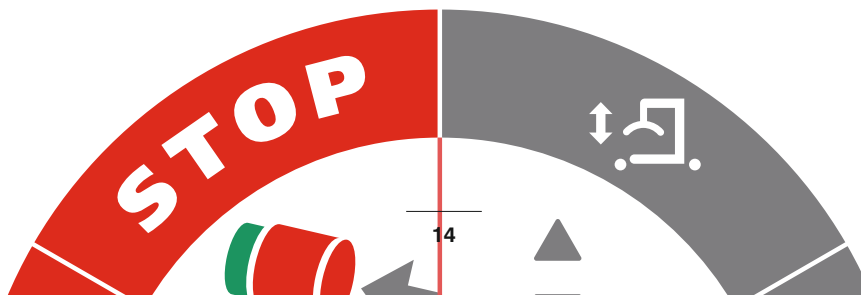

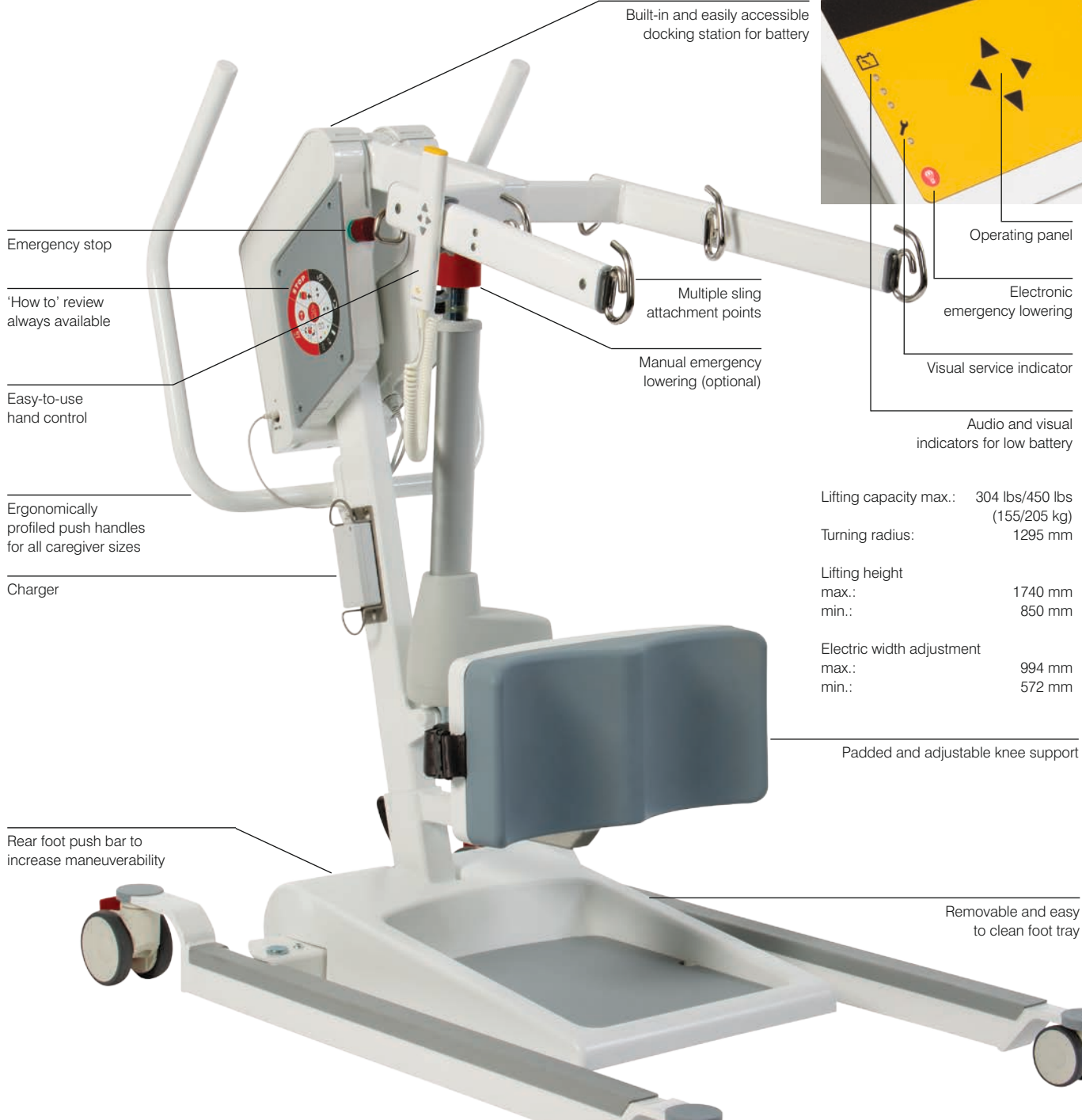

Emergency stop

'How to' review always available

Easy-to-use hand control

Ergonomically profiled push handles for all caregiver sizes

Charger

Rear foot push bar to increase maneuverability

Built-in and easily accessible docking station for battery

Multiple sling attachment points

Manual emergency lowering (optional)

Operating panel

Electronic emergency lowering

Visual service indicator

Audio and visual indicators for low battery

Lifting capacity max.: 304 lbs/450 lbs (155/205 kg)

Turning radius: 1295 mm

Lifting height max.: 1740 mm
min.: 850 mm


Electric width adjustment max.: 994 mm
min.: 572 mm

Padded and adjustable knee support

Removable and easy to clean foot tray



Time to care

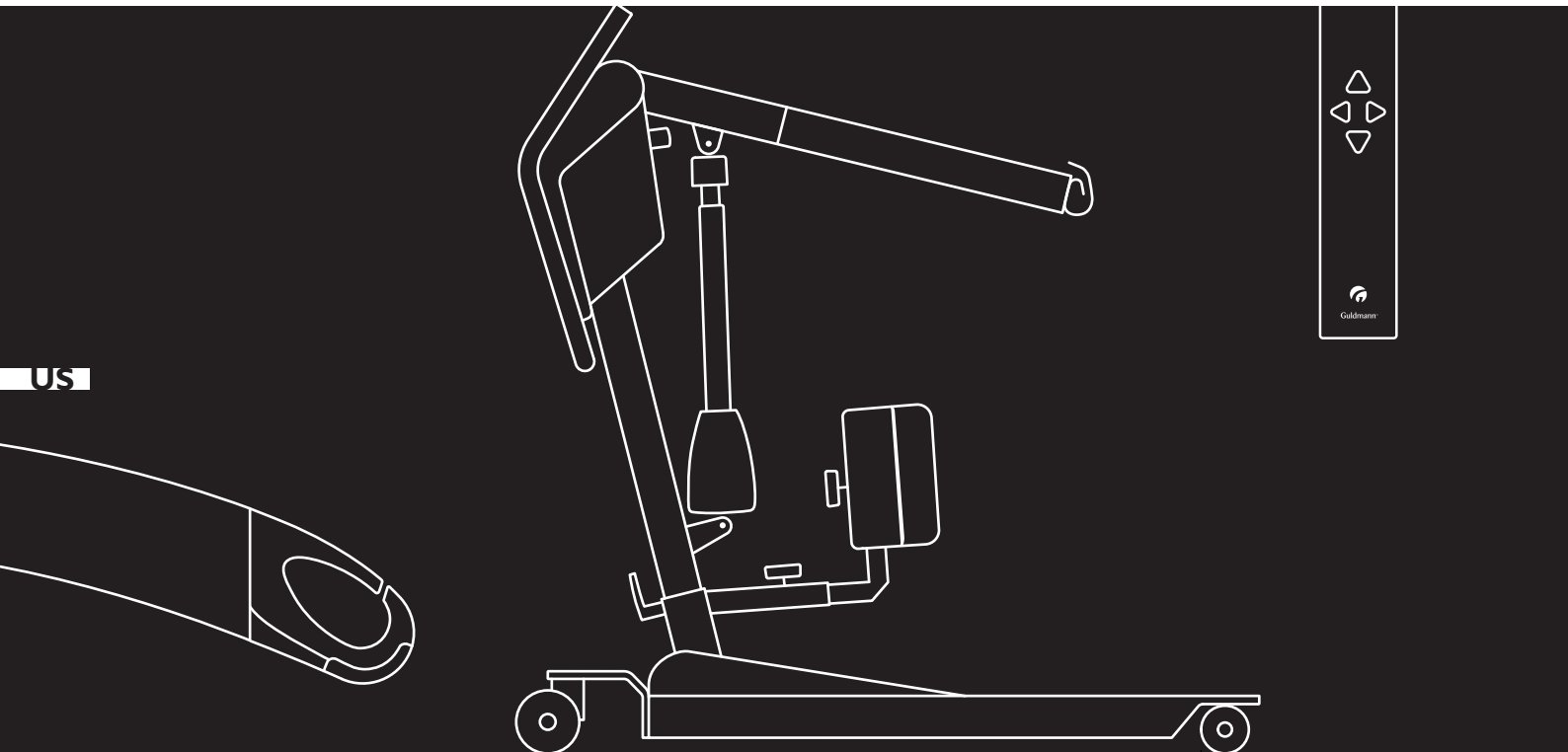

Exclusive Philippine Distributor:

WellnessPRO
tomorrow's medical devices

1 b]h, \$, F]Ma cbXYDUHŁGUbA][i Y' 5j YWf@ci fXYgGrž
Cffj[Uj 7h" DŁj][City 1605 Philippines
Mobile no.: +63 922 896 0964
Tel. no.: +63 2 636 3580, +63 2 637 0930, +63 2 637 0473
Fax: +63 2 636 7542
Email: wellnessproinc@gmail.com, jcjuan@wellnessproinc.com
www.wellnessproinc.com

Guldmann
Care-Lift Solutions ULC
155-99 Scurfield Blvd
Winnipeg, Manitoba, Canada
R3Y 1Y1
Toll Free 1 877 339 7779

© Guldmann US-2023/02/2014

Guldmann™